

Rauland Recommended Cleaning and Disinfecting Procedures

Rauland is committed to supporting your initiatives to provide the clean and sterile environments for today's patient care. Rauland recommends using the following cleaner solutions (in proper dilutions) on Rauland Responder® products.


Rauland Approved Cleaning and Disinfectant Solutions¹:

- Triple S Sanotracin RTU™
- JohnsonDiversey Alpha HP™/MC
- Virex II™ 256
- Clorox Healthcare® Bleach Germicidal Cleaner



These products can be safely used to clean, deodorize, and/or disinfect Responder Consoles, Station and Module Faceplates, Pillow Speakers, Call Cords, and all other Responder hardware products.

Both spray-and-wipe and soak tests has shown that the high-impact molded plastic and screening used to make these components will not be degraded if any of these cleaners are used as directed over prolonged periods of time. Equipment should always be cleaned using the manufacturer's recommended dilution on a well-wrung sponge, cloth, handpad or wipe and never spraying solution directly onto components.

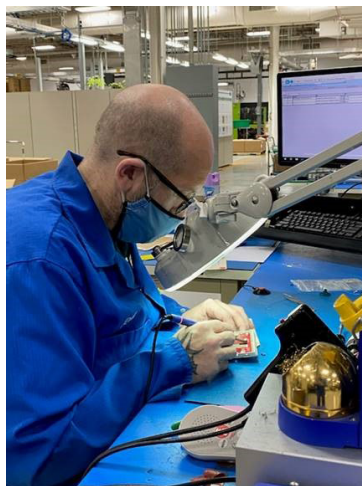

Rauland Service and Support

Safety and quality are our top priority at Rauland. Our production teams take pride in their work and surroundings. We adhere to 6-feet physical distancing, and Personal Protective Equipment (PPE) is provided to all team members in our factory. Product coming in from the field for repair are fully sanitized with Clorox Healthcare disinfecting wipes ensuring all viruses and bacteria are removed prior to repair work.

1 Rauland cannot recommend using JohnsonDiversey Tb™, as tests revealed component cracking and peeling.